


# H1

## CLINICAL CONNECTIONS & SETTINGS GUIDE

H1 controls direct pattern recognition command of multifunction grips for i-Limb Quantum\* or VINCENTevolution3 where no powered wrist or elbow is present. (\*with serial numbers M9618 or newer)

### PART 1: PHYSICAL CONNECTIONS

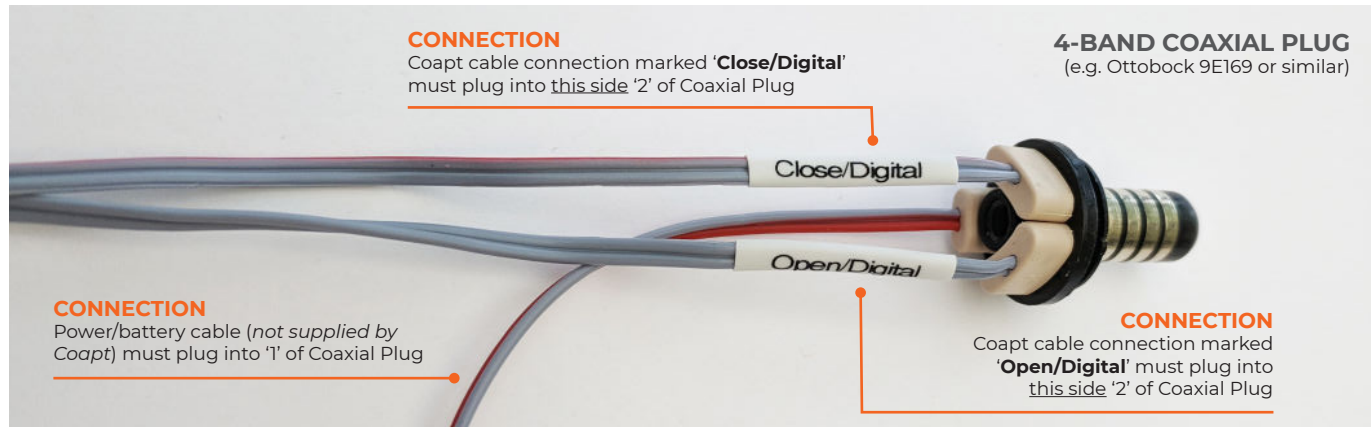

### PART 2: SOFTWARE SETTINGS

#### HANDS

##### I-LIMB QUANTUM\*

Ensure hand settings are the factory defaults.

For proper operation of Gesture Control via Coapt, ensure 'CC' is selected as the i-mo trigger in the i-Limb app

##### VINCENTEVOLUTION3

Ensure hand settings are the factory defaults