


H1

CLINICAL CONNECTIONS & SETTINGS GUIDE

H1 controls direct pattern recognition command of multifunction grips for i-Limb Quantum* or VINCENTevolution3 where no powered wrist or elbow is present. (*with serial numbers M9618 or newer)

PART 1: PHYSICAL CONNECTIONS

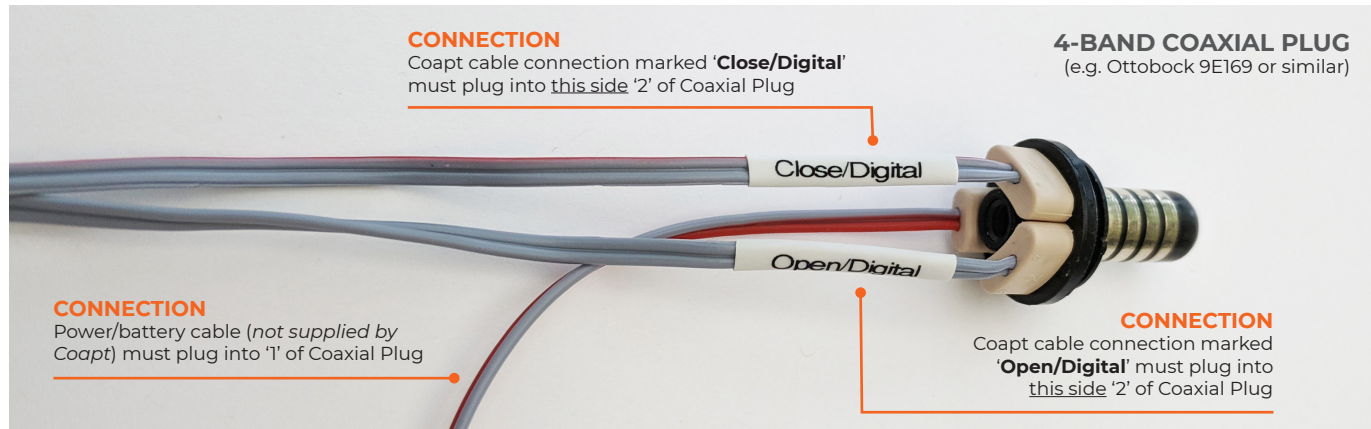

PART 2: SOFTWARE SETTINGS

HANDS

I-LIMB QUANTUM*

Ensure hand settings are the factory defaults.

For proper operation of Gesture Control via Coapt, ensure 'HO' is selected as the i-mo trigger in the i-Limb app

VINCENTEVOLUTION3

Ensure hand settings are the factory defaults