


18

CLINICAL CONNECTIONS & SETTINGS GUIDE

18 controls pronation/supination of Motion Control “6-band” powered wrist and direct pattern recognition command of multifunction grips for the COVVI Hand.

PART 1: PHYSICAL CONNECTIONS

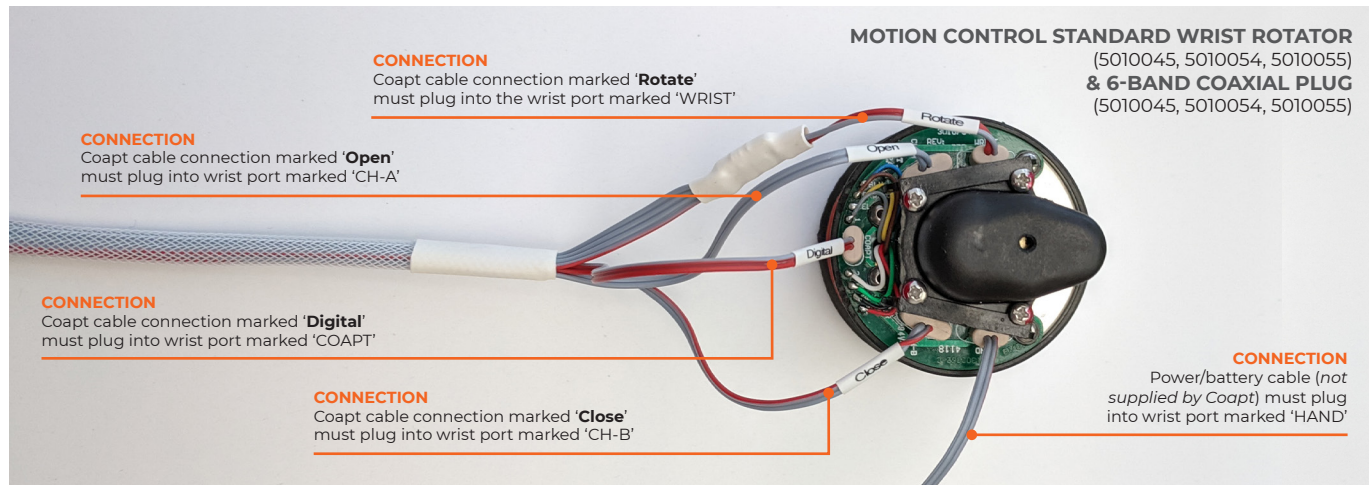

PART 2: SOFTWARE SETTINGS

WRISTS

MOTION CONTROL STANDARD WRIST ROTATOR & 6-BAND COAXIAL PLUG

No settings required

HANDS

COVVI HAND

Ensure all “Triggers” in the Grip Switch Trigger settings in the COVVI Go app are set to “Trigger Off”

Ensure all other hand settings in the COVVI Go app are set to the factory defaults

