

JO

CLINICAL CONNECTIONS & SETTINGS GUIDE

JO allows pattern recognition command of Ottobock AxonArm Ergo electric lock with optional AxonRotation and the AxonHook and/or Michelangelo Hand.

PART 1: PHYSICAL CONNECTIONS

OTTOBOCK AXONARM ERGO

CONNECTION
Coapt cable connection marked '**CONNECT TO Elbow at Input**' must plug into the **Input** port on the AxonArm Ergo

CONNECTION
Coapt cable connection marked '**CONNECT TO AxonMaster**' must plug into blue/gray cable from AxonMaster

CONNECTION
Cables marked '**1**', '**2**', and '**3**' plug into corresponding inputs of the Ottobock AxonMaster

PART 2: SOFTWARE SETTINGS

ELBOW

OTTOBOCK AXONARM ERGO

Ensure all settings in the Ottobock Data Station 'Customizing' screen are set exactly as shown.

WRIST

OTTOBOCK AXONROTATION (OPTIONAL)

Ensure all settings in the Ottobock Data Station 'Customizing' screen are set exactly as shown.

HANDS

OTTOBOCK MICHELANGELO HAND, AXONHOOK

Ensure all settings in the Ottobock Data Station 'Customizing' screen are set exactly as shown.