


**L1**

# CLINICAL CONNECTIONS & SETTINGS GUIDE

*L1 allows pattern recognition command of Mobius Bionics LUKE Arm Radial Configuration powered 2 DOF wrist and hand open/close with additional grip cycling.*

## PART 1: PHYSICAL CONNECTIONS

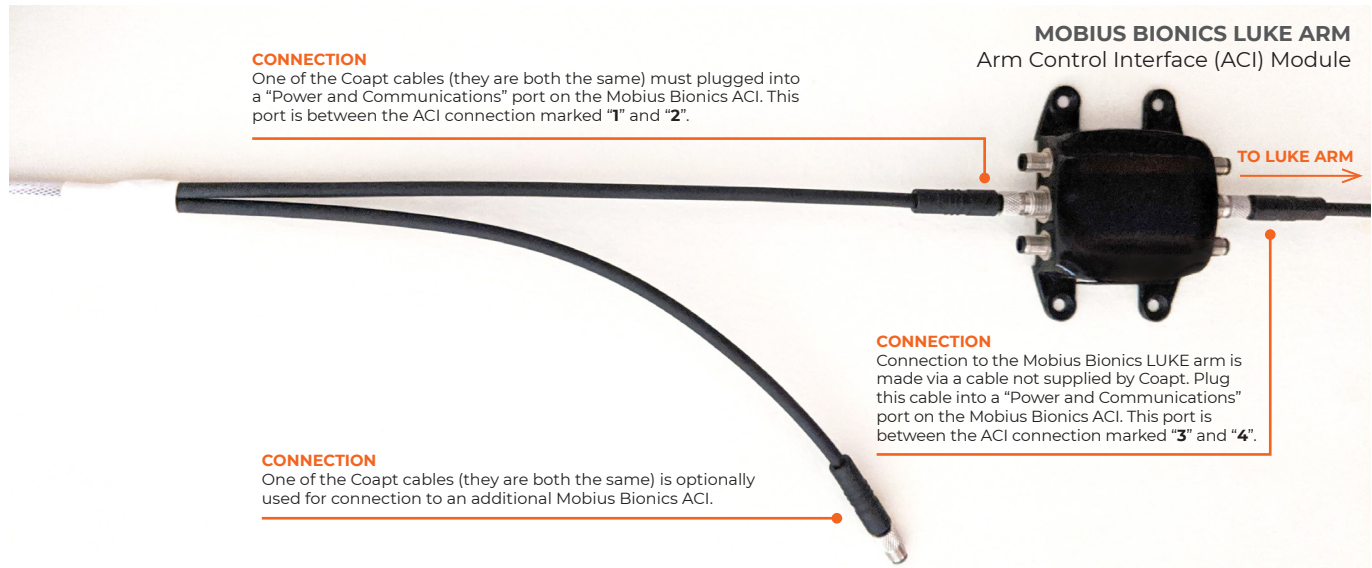

## PART 2: SOFTWARE SETTINGS

### WRIST, HAND

#### MOBIUS BIONICS LUKE ARM RADIAL CONFIGURATION

Two possible pre-set program configurations are available from Mobius Bionics. Contact Mobius to access either program file.

1) "COAPT RC - Grip on COAPT"

Control of hand grip selection is performed by Coapt pattern recognition.

2) "COAPT RC - Grip on ACI"

For use with external Mobius Bionics switching accessories for the control of hand grip selection.

**Note:** Accessories will be connected to Channels "3" and "4" on a LUKE Arm Control Interface (ACI)